


# AIR N GO

EASY

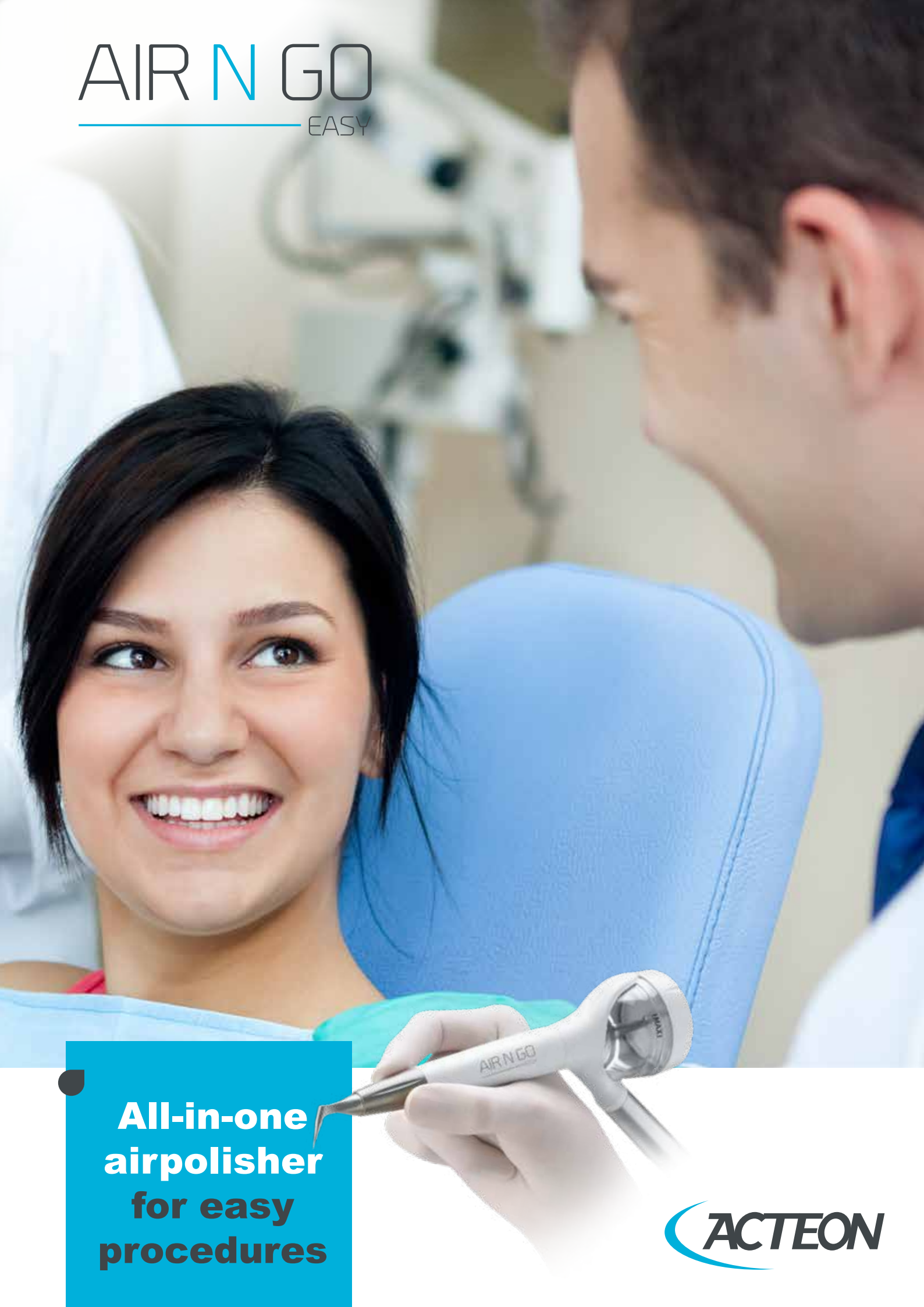

**All-in-one  
airpolisher  
for easy  
procedures**

**ACTEON**

# Simplicity and efficacy all-in-one



For a supra- and a sub-gingival treatment



Dual function

- Supra-gingival polishing **SUPRA Function** : daily prophylactic polishing.
- Sub-gingival polishing **PERIO function** : maintenance, periodontal and peri-implant treatments.

This medical device allies simplicity with performance.

A complete solution

- 4 nozzles for active control of the progress of periodontal disease



- Depending on your clinical needs, choose:
  - Sodium bicarbonate-based **CLASSIC** powders
  - Glycine-based **PERIO** powder



A single handpiece

## A single handpiece to cover a range of care and treatment

AIR-N-GO® easy combines essential functions with reliability to facilitate your everyday life:

- Fast start-up: direct connection to the high speed handpiece connector.
- Greater freedom of movement: 360° rotation, fast and accurate treatment of areas requiring care.
- Easier cleaning of the handpiece: main parts can be dismantled.



Facilitated maintenance

# 4 allies to prevent and fight periodontal disease



## Air-N-Go® SUPRA nozzle

- Tool suitable for any supra-gingival prophylactic treatment in combination with AIR-N-GO® **CLASSIC** powders
- Guarantees a fast, efficient procedure for greater working comfort



Dr P. ROUAS (France) clinical case



## Air-N-Go® PERIO MAINTENANCE nozzle

- Reduced consumption of AIR-N-GO® **PERIO** fine powder particles in the maintenance phase
- Treatment of mucositis and return to a healthy clinical situation
- Action against the progression of periodontal disease



Dr C. LIM (U.K.) clinical case



## Air-N-Go® PERIO EASY nozzle

- Innovative design in a "shoehorn" shape providing easy sub-gingival insertion
- A real asset for non-surgical, sub-gingival treatment of teeth and implants (3-8mm periodontal pockets)
- 5 seconds per face is all that is needed



Dr C. LIM (U.K.) clinical case

AIR-N-GO® nozzles, which result from the most advanced fluid dynamics modelling techniques, provide continuous, accurate and controlled spray whatever the clinical context.

In daily practice these nozzles provide power and efficiency for the removal of supra- and sub-gingival biofilm.



## Air-N-Go® PERIO nozzle

- Sub-gingival use for surgical and non-surgical treatments (tunnelling and open flaps)
- Powerful double lateral spray
- Decontamination of deep periodontal pockets (8-10mm)



Dr A. PATEL (U.K.) clinical case

# Customized solutions for your clinical needs

AIR-N-GO® powders have been specially designed and studied to produce effective treatment that is gentle on all treated surfaces. All these powders are compatible with the AIR-N-GO® easy, no change of handpiece is required.



5 fresh flavours based on 100% natural aromas or essential oils:  
 Bottles (4 x 250g): F10254 Neutral, F10253 Cola, F10252 Raspberry, F10250 Peppermint and F10251 Lemon  
 Sticks (50 x 20g): F10260 Neutral, F10259 Cola, F10258 Raspberry, F10256 Peppermint and F10257 Lemon

## Air-N-Go® CLASSIC powders

Sodium bicarbonate-based  
 (≈ 76 µm)

- Complete range of active **supra-gingival powders** which meet any patients' preferences
- Scrupulous cleaning giving instant results
- Action of removing coloured stains and bacterial plaque, even in areas that are difficult to access
- Better patient comfort: gentle and efficient with limited bleeding during treatment




F10125  
 Supplied as standard

Used with AIR-N-GO® EASY SUPRA 120°, indications are:


- Finishing after scaling,
- Preparation for whitening,
- Cleaning of inter-proximal areas, fissures and crevices,
- Preparation of the tooth surface before etching,
- Cleaning of orthodontic brackets,
- Removal of supra-gingival biofilm, plaque, stains and temporary cement residues

### COMPETITOR

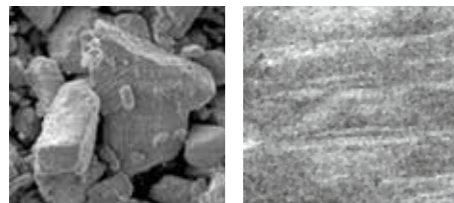

Stratified particles with aggressive geometry (x1000) | Treated enamel shows numerous scratches (x300)

### AIR N GO

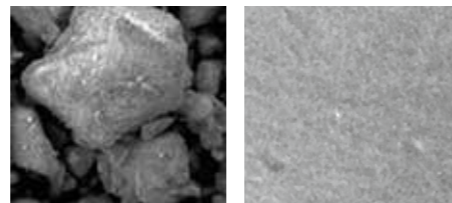

Particles with softened geometry (x1000) | Treated enamel: surface preserved without impacts or scratches (x300)

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

## Air-N-Go® PERIO powder

Natural glycine-based  
 (≈ 25 µm)

- Exclusive use for **sub-gingival treatments** as a complement to the initial therapy
- Low abrasiveness preserving the treated surfaces (teeth and implants) and the most fragile anatomical areas
- Prevents the progression of the sub-gingival biofilm
- **Multiple advantages with unique properties:**
  - Plaque removal: during periodontal maintenance on exposed root surfaces without sacrificing substance
  - Reduction of the bacterial load in the periodontal pockets



F10270 (3 x 160 g)



F10126  
 Supplied as an option

Used with AIR-N-GO® PERIO EASY, indications are:

- Removal of the sub-gingival biofilm
- Treatment of periodontal disease and peri-implantitis (Pockets 3mm - 8mm)



F10127  
 Supplied as an option

With AIR-N-GO® PERIO, the indication is:


- Treatment of periodontal disease and peri-implantitis (Pockets 8mm - 10mm)



F10127  
 Supplied as an option

With AIR-N-GO® PERIO MAINTENANCE, indication is:

- Periodontal maintenance (Pockets up to 4mm)

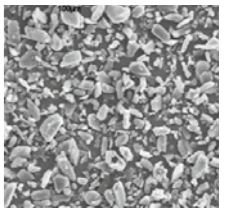

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos  
 AIR-N-GO "PERIO" (x1000)



ACTEON

# in your daily practice



## Prophylaxis

### Diagnosis

Reveal the diseases linked to oral care thanks to Soprocare®

### Treatment

Newtron® / Air-N-go®, Bliss in synergy to combine efficacy and comfort for a precise and efficient treatment

### Follow up

Have a reliable and immediate check of the clinical condition



AIR N GO  
EASY



SOPROCARE



NEWTRON



Bliss  
by Acteon

For more information about good practice and the use of AIR-N-GO® easy, see our [YouTube](#) channel



Air-N-Go® Easy: Dental air-polishing handpiece - Newtron® P5 XS B.Led: Dental Ultrasonic Control Console Class IIa medical devices - CE0459 (GMED) - Manufacturer: SATELEC® - France - Soprocare®: Class I medical device- CE - Manufacturer: SOPRO® - France Bliss: Class I medical device - CE - Manufacturer: PRODONT HOLLIGER - France Read carefully the instructions for use available on [www.acteongroup.com](http://www.acteongroup.com) For professional dental use only - Updated on: 07/2021

SATELEC® | A Company of ACTEON® Group  
17 av. Gustave Eiffel | ZI du Phare | 33700 MERIGNAC | FRANCE  
Tel + 33 (0) 556 340 607 | Fax + 33 (0) 556 349 292  
E-mail: [info@acteongroup.com](mailto:info@acteongroup.com) | [www.acteongroup.com](http://www.acteongroup.com)

