

CAVEX ImpreSafe Impression Disinfectant



Bactericide, fungicide, virucide disinfectant suitable for disinfecting alginate, polyether and silicone impressions

- Aldehyde free and has a neutral smell
- **Kills bacteria, fungi and viruses in 3 minutes**
- Highly concentrated saves storage space
- Fast, safe, non-toxic and effective impression disinfectant
- No negative influence on the impression surface
- The kit includes a disinfection container, one litre of disinfection concentrate and a digital timer




1) Squeeze bottle and fill measuring cup up to 30ml



2) Pour water in reservoir first, up to indicated stripe.



3) Pour disinfectant into reservoir.

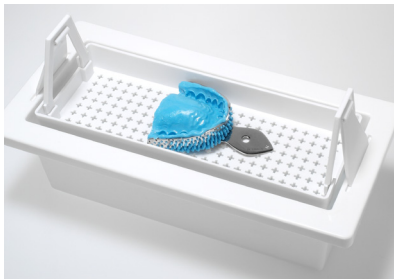

4) Place impression on tray in container.



5) Lower impression into fluid. Disinfect for 3 minutes by using the timer.



6) After 3 minutes, take the tray out of disinfectant fluid.



7) Rinse off with water.



8) Shake off excess water and put impression in a plastic bag. Seal airtight. Mark the bag as "disinfected".

Introductory Starter Kit

- | | | |
|----|---|-------------------|
| 1 | X | Cavex ImpreSafe |
| 1 | X | Disinfectant Bath |
| 1 | X | 3 Minute Timer |
| 50 | X | Transport Bags |

1 Litre Bottle = 33 Weeks of Disinfectant*

*Normal Use