


EZ-TEST

BIOLOGICAL INDICATORS

INTERPRETATION GUIDE

GETINGE

GETINGE GROUP

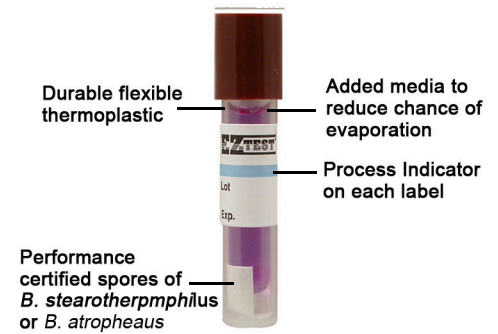

NEGATIVE CONTROL



POSITIVE CONTROL

1. Examine the indicator at regular intervals for any colour change (ie. 18 hours, 24 hours). The appearance of a yellow colour indicates bacterial growth. No colour change indicates adequate sterilization.
2. Act on a positive test (a colour change of yellow) as soon as the colour change is noted. Notify the appropriate personnel (i.e Infection Control). Always retest the sterilizer with several EZTest indicators throughout the test load. EZTest indicators can be subcultured if identification of positive growth is desired. Recommended subculturing procedure techniques are available upon request.
3. The recommended incubation time is 24 hours
4. Record the results