

## **TATS® DENTURE CLEANER**

Finally an effective denture cleaning solution. TATS® DENTURE CLEANER is a reusable solution which has been developed & successfully tested by specialists with more than 50 years of experience in the dental industry. TATS® removes plaque & stains, such as those left by nicotine, tea & coffee. TATS cleans the full upper and lower dentures, part metal dentures & orthodontic appliances. TATS® is safe to use & does not bleach or alter the characteristics of the plastic denture base. TATS does not oxidize or damage metal attachments. TATS® is biodegradable & an environmentally friendly product.

### **DIRECTIONS FOR USE**

- For initial use & badly stained dentures, soaking overnight in TATS® solution is strongly recommended.
- Rinse dentures daily & soak in denture bath with TATS® DENTURE CLEANER for at least 20 minutes. After soaking, rinse & scrub the dentures under running water, using a suitable denture brush to remove all traces of TATS® DENTURE CLEANER, before replacing in mouth.
- Do not mix with toothpaste or other cleaning agents.
- Replace lid on denture bath after use, leaving TATS® DENTURE CLEANER in the bath for the next soaking. It is recommended that the solution be replaced every week. Shake well before use. Avoid spillage on the bench top & floor coverings as staining may occur.