VibraKleen™ E²

Enzyme Ultrasonic Cleaner

KULZER

MITSUI CHEMICALS GROUP

Gebrauchsinformation

Instructions for use

Mode d'emploi

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Kulzer, LLC
4315 South Lafayette Blvd.
South Bend, IN 46614-2517
1-800-431-1785
www.KulzerUS.com

40000210 R 7/17

VibraKleen™ E²

Instructions for use

GB

Enzyme Ultrasonic Cleaner
Effervescent Tablets

DIRECTION FOR USE: For general purpose ultrasonic cleaning, evacuation system cleaning and pre-soak. TWO TABLETS MAKE 4 LITERS. Add two tablets to 4 liters of warm tap water. For 2 liters or less, use one tablet. Do not mix with any other cleaning or disinfecting product. Wear eye protection and rubber gloves while handling.

FOR ULTRASONIC CLEANERS: Depending on size of ultrasonic unit, drop either one or two tablets directly into tank. Fill with warm tap water. Water temperature should not exceed 60°C. If pre-mixing for later use, place two tablets in 4 liter container and fill with warm tap water. Store mixed solution in a covered container until ready to use. Ultrasonically clean items for 10 minutes. Extended time in solution can promote rusting on some types of steel or carbides. Rinse items thoroughly. Dry instruments before proceeding with the sterilization process. Replace solution when it becomes visibly cloudy, or minimally, once a day as recommended by the "ADA Recommendations for Dental Office Procedures".

FOR EVACUATION SYSTEMS: At the end of each day, add two VibraKleen E² tablets to 4 liters of warm tap water. Draw at least 1 liter of solution through each suction outlet and run system for a few minutes as recommended by the manufacturer. If heavy surgical procedures are performed, flush lines several times daily.

FOR INSTRUMENT PRE-SOAK: Soak instruments in solution for at least 10 minutes, longer for tougher stains. Do not exceed temperature of 60°C. Do not leave instruments in solution for longer than 24 hours.

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Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as and to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control, and therefore, entirely your own responsibility. Should, in spite of this, liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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