

ASEPTI ACTIVE WIPES

Hospital grade disinfectant

ARTG 340994

Asepti Active Wipes is a hospital grade disinfectant approved for single step cleaning and disinfection of dirty surfaces.

Asepti Active Wipe is a broad spectrum bactericide effective against VRE & MRSA and has virucidal efficacy against Human Coronavirus, COVID-19 [SARS-CoV-2] and Influenza A.



DISINFECTANT CLEANER

Asepti Active Wipes contains a combination of quaternary biocides - Benzalkonium Chloride & Dialkyldimethylammonium chloride - designed to provide optimal efficacy against QAC resistant organisms.

CONDITION	EXAMPLE	CONTACT TIME
Broad Spectrum bactericidal efficacy including	Pseudomonas aeruginosa; Proteus vulgaris Escherichia coli; Staphylococcus aureus	8 minutes in dirty conditions
Bacterial Outbreak event Antibiotic Resistant bacteria	Vancomycin Resistant; Enterococcus faecalis - VRE; Methicillin Resistant; Staphylococcus aureus - MRSA	5 minutes in dirty conditions
Viral Outbreak event	Human Coronavirus COVID-19 [SARS-CoV-2] severe acute respiratory syndrome coronavirus 2; Influenza A	5 minutes in clean conditions 10 minutes in dirty conditions

Asepti Active Wipes are easy to use, they ensure the correct concentration of chemistry is available in each application and reduce the risk of cross contamination.

Asepti Active Wipes are conveniently packed, low lint, low foam and are available in a range of pack sizes to suit varying applications.

Asepti Active Wipes are compatible with **Asepti Active Liquid**.

A range of dispensers to suit the flat pack and pail format are also available encouraging accessibility and promoting usage.

DIRECTIONS FOR USE

- ▲ Wear gloves and additional PPE where required.
- ▲ **Asepti Active Wipes** are safe for use on most surfaces. They are not suitable for therapeutic devices or as a baby wipe.
- ▲ **Asepti Active Wipes** are single use only, supporting infection control protocols.
- ▲ Do not dispose in toilets.

SAFETY INFORMATION

Do not mix with any other products. For further information and safety instructions see product label and Safety Data Sheet.

Product Code	Product Description	Pack Size
7101608	Asepti Active Wipes	12 x 100 wipes
99960090	Flat Pack Holder	Each
7100096	Asepti Active Wipes	3 x 325 wipes
9996-1208	325 Wipe Wall Bracket	Each
HC-00023	Wall chart - Asepti Active Liquid & Wipes	Each

PLEASE SPEAK TO YOUR HEALTHCARE MANAGER OR CALL **1800 022 002** FOR FURTHER INFORMATION